

Prime Cement Mold System

プライムセメント
モールドシステム

Taper Wedge Shaped Spacer
Preserve Bones




Speciality

Most suitable for post stem removal
Acetabulum formation
Confirmation of stem size by trial

Prime Cement Mold System

製品情報

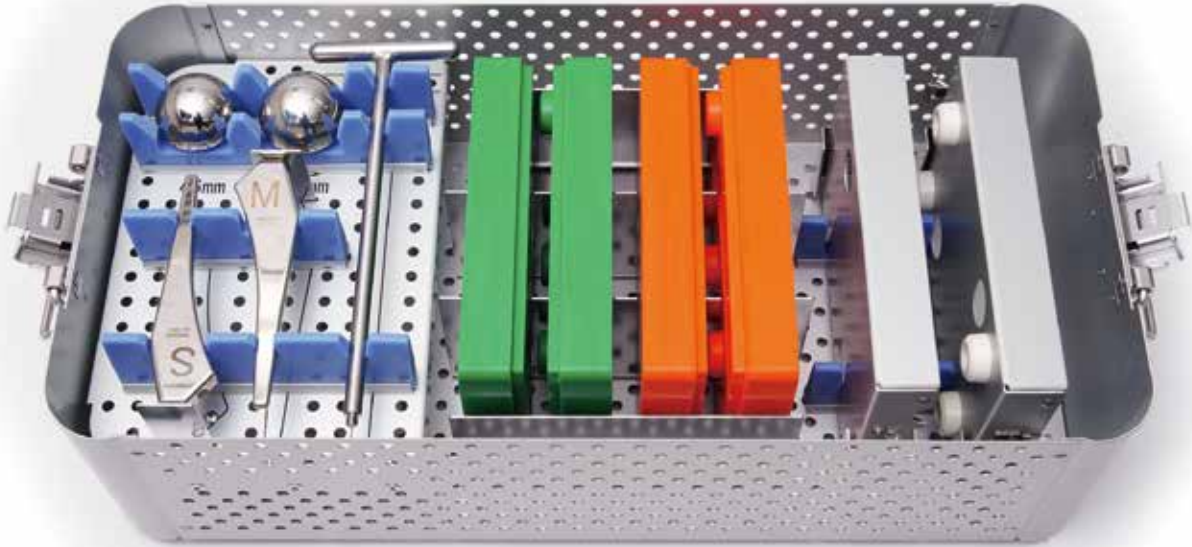

- Easy discrimination by each color
- Accurate size decision by interoperative trial
- Easy installation by taper-wedged shape
- Reusable by autoclave sterilization

Surgical Techniques



1

After size confirmation,
pour cement into both side of the case



2

Crimp the case,
wait until cement get stiffened



3

Locate the cement mold



- Pour the cement after location of holes of the case (easy to remove the cement from distal)
- Bone cement is not attached
- Two packs of bone cement needed.

This product is not an artificial joint. It might be broken under load.



How to use: <https://youtu.be/GqOcQERBymw>

製品諸元

Size Chart

Size (mm)	S(Orange)	M(Green)
Head	42	45
Offset	34.6	37.4
Stem	142	149.3
Proximal AP width	11	11.5
Proximal ML width	25.6	27.2
Distal ML width	6.5	7.9
Neck shaft angle	130°	130°

Order Info

セット内容	カタログ番号
Mold	S PM0400S
	M PM0400M
Mold case	PM05000
Cup trial	S PM02046
	M PM02049
Stem Trial	S PM0300S
	M PM0300M
T handle	PM01000